


- Cheek Shield
- Flexible Isthmus
- Dual Vacuum Channels
- Illumination Path
- Tongue and Throat Shield
- Integrated Bite Block with Rear Evacuation Channel

96-0801 Rev C


1 Mouthpiece Attachment

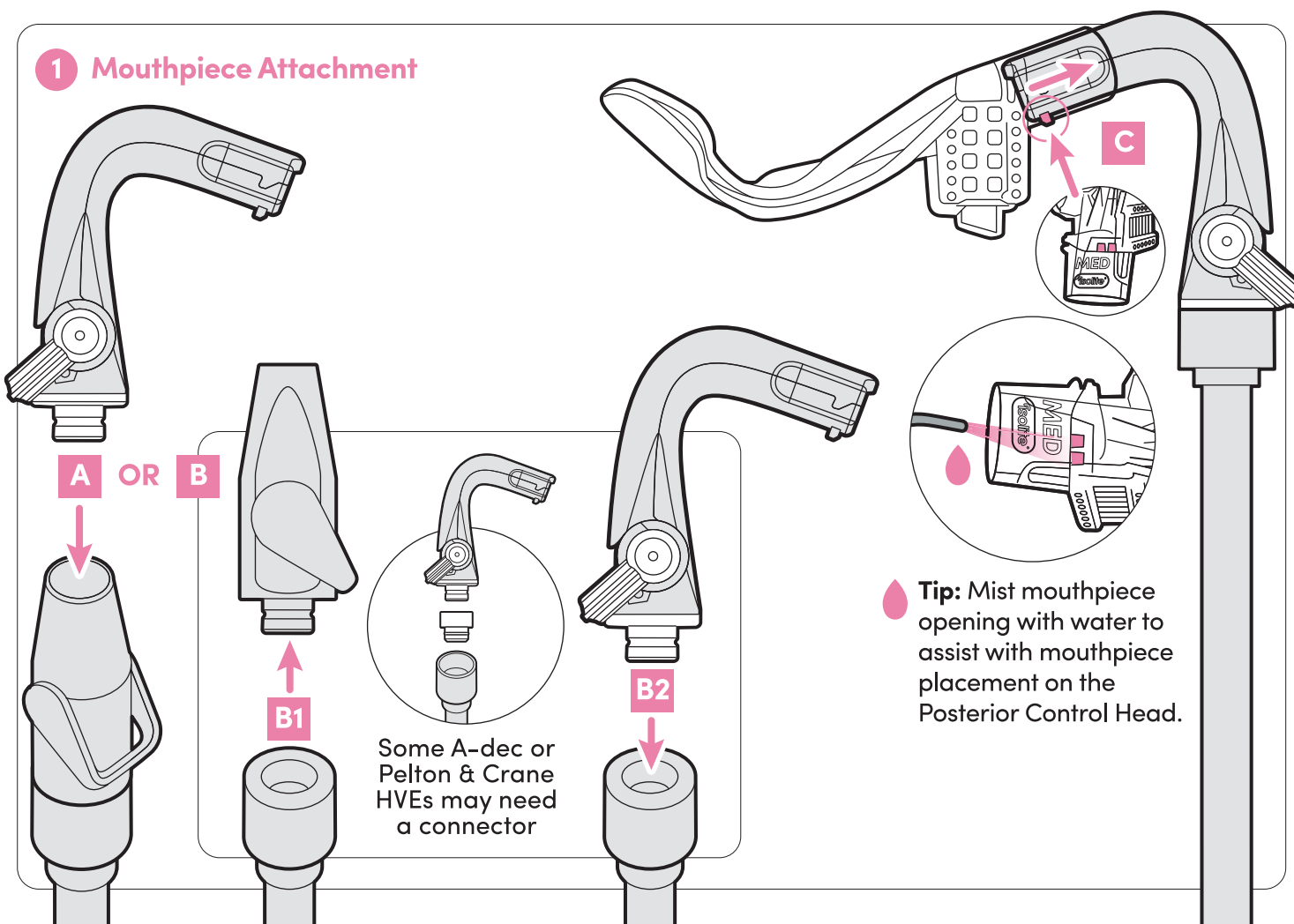


2 Mouthpiece Lubrication

Mist water on the back side of mouthpiece before insertion.

You can also lubricate patient's lips for comfort.




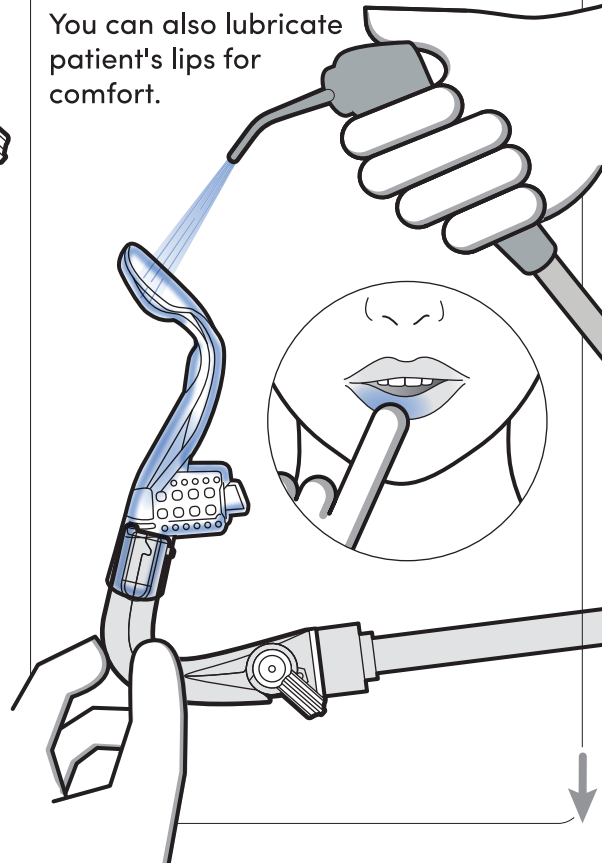


3 Mouthpiece Insertion

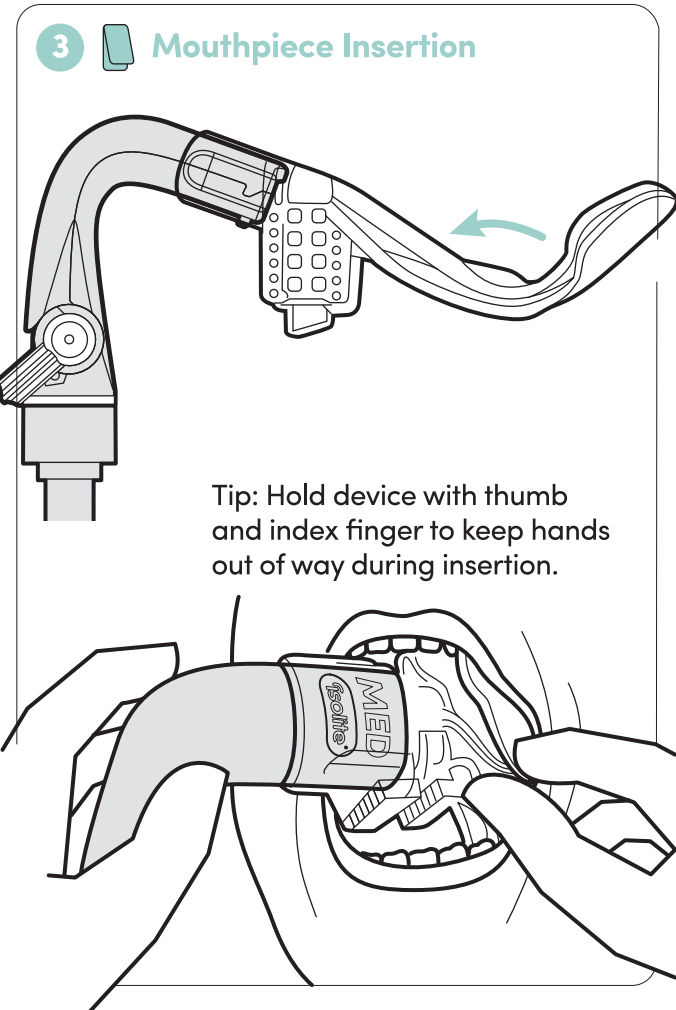


4 Mouthpiece Insertion

Insert into cheek.

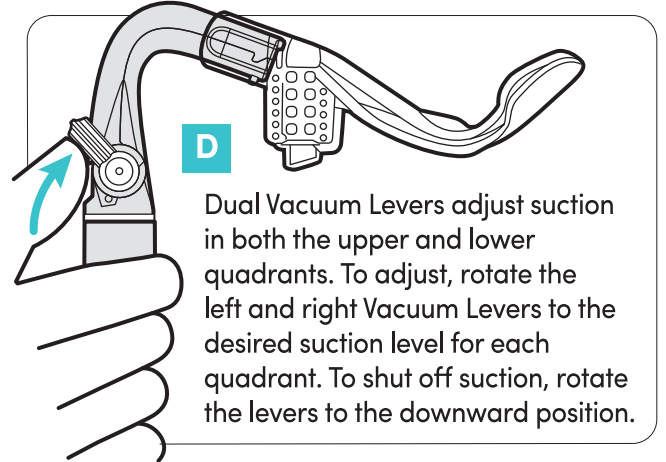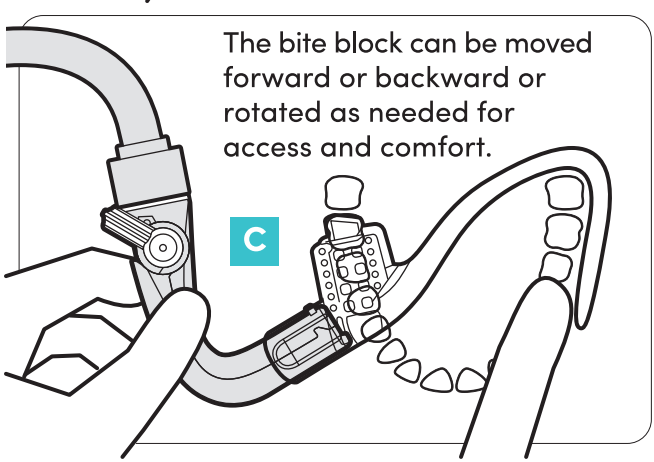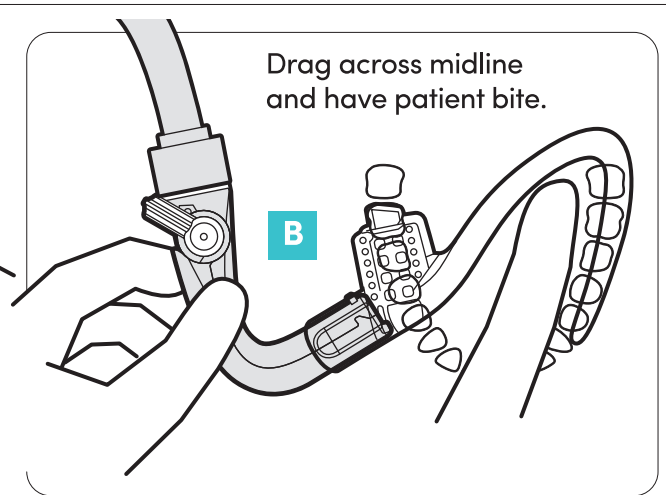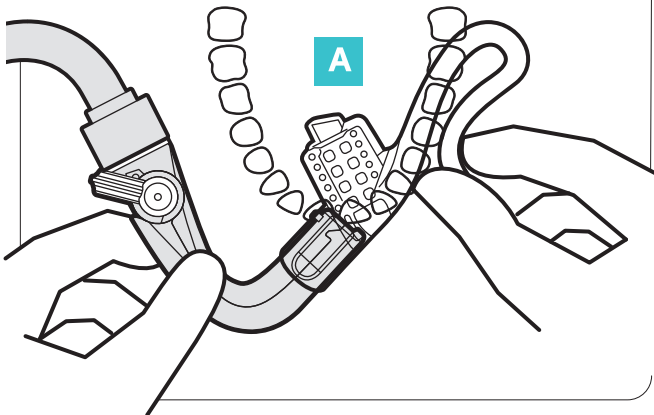


5 Intended Use

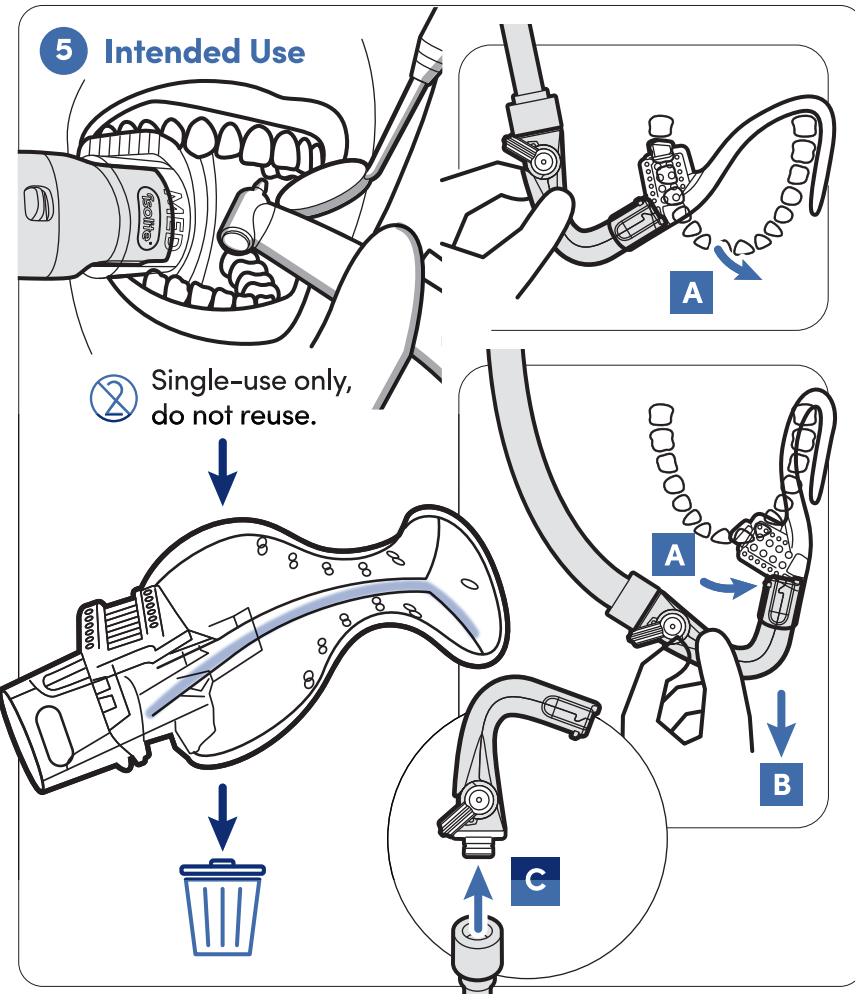

6 Cleaning, Disinfection and Sterilization

Cleaning and Sterilization of Posterior Control Head

Zyris recommends cleaning and sterilization of the Posterior Control Head after each patient use. To clean the Posterior Control Head, remove levers and separate (A, B, C). Use brush with enzymatic cleaning solution to remove all visible biomatter (D). Rinse with water and place components in an autoclave pouch to be sterilized. Use validated sterilization cycles.

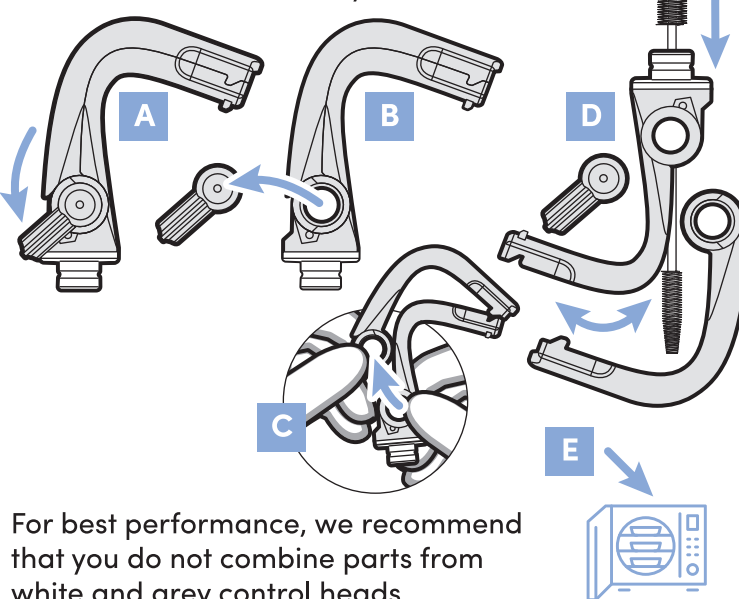

Periodic Maintenance

Apply provided lubricant to vacuum lever o-rings.

